

Chemdye® Splat Cleaning Performance Test

Materials



Chemdye Splat Cleaning Indicator + CDWAH Holder for Washer Disinfectors



Chemdye Splat Cleaning Indicator + CDWAH-U Holder for Ultrasonic Washers

Procedures

1 Take a clean, dry and empty Splat Holder. Place the indicator inside of it.




2 Place the holder with the indicator at one end of each basket of your washing machine together with the material to be cleaned. Make sure the indicator ink is oriented towards the center of the washing machine.





3 Place each loaded basket (indicator + material) in your washing machine and perform a regular washing cycle in order to monitor the efficacy of each washing cycle.
NOTE: The test can be also performed in an empty washing machine.

Results


Unused

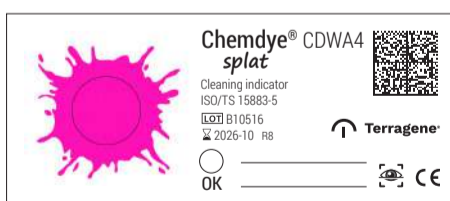

Failures (Any residual ink)

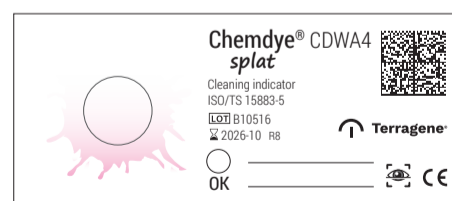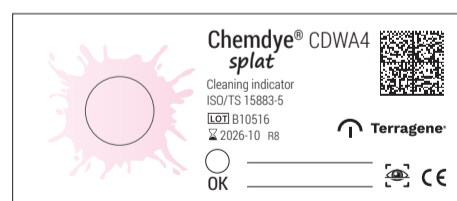

Pass

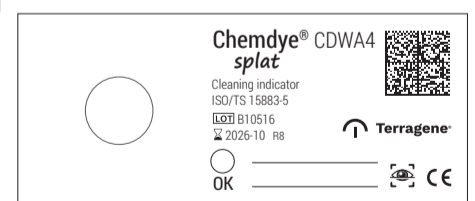

Possible reasons for system failure (for washer disinfectors):

Mechanical mainly related failure:
 › Low water pressure
 › Location inside the chamber / Overloaded chamber

Non-mechanical mainly related failure:
 › Low detergent dosage / concentration
 › Low Temperature
 › Low water quality (hardness, pH, non-treated)
 › Reduced exposure time



Let's work together to create a better future